# Litec 2G Plus

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**LIGHTWEIGHT WHEELCHAIR with / without drum brake**

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**EN****Content**

Parts of wheelchair	3
Introduction	10
Declaration of conformity	10
Indication	10
Delivery content	10
Safety instructions	10
Assembly	12
Mounting wheels	12
Assembling footrests	12
Adjusting footrests	12
Adjusting seat	12
Armrests	13
Parking brake	14
Hand brake	14
Using the tilting aid	15
Folding wheelchair	15
Cleaning/ Maintenance/ Reuse	15
Warranty	15
Label	28
Specifications	28

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1. Push handle
2. Swing-away armrest
3. Hand rims
4. Tip assist
5. Quick release axle
6. Parking brake
7. Steering wheel (front)
8. Footrests (plates)
9. Footrests (whole)
10. Unlock footrests
11. Back support
12. Adjusting forerum of the front wheel
13. Hand brake optional (wo. picture)

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## Introduction

Thank you for choosing a wheelchair by DRIVE MEDICAL. You won't be disappointed by the combination of design, functionality and quality of this modern and versatile wheelchair.

Please read this user manual carefully before using the wheelchair. You will find important safety instructions as well as tips on how to use and clean this wheelchair.

For questions or additional information, please contact a DRIVE MEDICAL specialized dealer, that provided this wheelchair.

This user manual includes all necessary and essential guidelines for adjusting as well as using the wheelchair Litec 2G Plus.

This user manual can also be found online or requested of the customer service. In doing so, you can find enlarged versions for visually impaired people. Blind people must be instructed by an accompanying person.

## Declaration of conformity

The DRIVE MEDICAL GMBH & Co.KG declares conformity for this Standard Wheelchair Litec 2G Plus according to the EG-guidelines 93/ 42 for medical products.

## Indication / Contra-indication

This wheelchair must only be used as transportation or mobilization of handicapped people, indoors and outdoors on all surfaces.

**The use of this wheelchair is inappropriate when:**

- Perceptual disorder
- Vestibular disorder
- Limb loss of both arms
- Joint contracture/ Damaged joints on both arms
- Unable to sit

## Scope of delivery content

1. Standard-wheelchair Litec 2G Plus
2. User Manual

## Safety instructions

For proper and safe use, we advise you to regard the following instructions:

### STABILITY AND BALANCE

This wheelchair was designed to ensure stability, that is needed in ones every day use.

Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. We advise you to wear the seatbelt, to increase your safety.

### MAXIMUM CAPACITY

The approved maximum capacity of the user is 125 kg. But the level of activity is an essential factor. For example: An active user weighing 75 kg can stress the wheelchair more than a user weighing 100 kg. There for we advise you to contact a DRIVE MEDICAL specialized dealer before buying a wheelchair, that which will be used on a daily basis.

### REACHING AN OBJECT FROM THE WHEELCHAIR (SITTING)

The balance limit to reach an object was set by a representative group of wheelchair users.

- Only the arms may be extended over the wheelchairs seat.
- The body and head should stay inside the limits of the seat.

### LEANING FORWARD

Don't lean over the armrests with your body. You must lean on something and bend down to reach an object in front of you. Due to this, you can use the steering wheels in the front as an aid (by leaning forward), to maintain stability and balance. The correct alignment of the wheels is essential to your safety.

### LEANING BACKWARDS

Set your wheelchair as close as possible to the desirable object, that you can easily reach it by extending your arms, while sitting in a normal position. You must not lean back, for the wheelchair may tip over.

### LATERAL TRANSFER

This transformation can be performed alone, assuming you are agile and strong enough.

- Set the wheelchair as close to the place, which you want to sit on, as possible, whereby the steering wheels point forward. Activate the fixing brake and move your body towards the desired place.
- If there's no support beneath you while moving, please use a board if possible.
- If you are capable of standing alone, and if your torso is strong and agile enough, you may change your position by yourself. Fold the footrests up and put them aside to make room. Bend your body forward, while leaning on the two armrests, and stand up. Then move your body toward the place, where you want to sit, while distributing your weight to your arms and hands.

## FORWARD TRANSFER

If you're capable of getting up and standing on your own, you may change seats by leaning forward. If you have trouble with this movement, we advise you to seek assistance from an accompanying person or nurse, to ensure an easy and safe transfer.

Activate both brakes and make sure both steering wheels point towards the front.

Please bear in mind that you must not support yourself on the brakes.

Grab the handle of the armrest firmly and support yourself on your arms, to stand up easily. The accompanying person may help you complete the transfer.

- Position yourself as close to the desired place as possible.
- When changing your position, please sit as far back as possible, to prevent screws from breaking, seat cushions from getting damaged, or the wheelchair from tipping over.
- Activate the two brakes; these brakes should in no way be used as support for the transfer.
- You must never support yourself on the footrests, while sitting or getting up.

## TIPPING OVER, HURDLING OBSTACLES, FOR EXAMPLE CURBSIDES

### Moving onto the sidewalk:

The accompanying person positions the wheelchair straight in front of the curbside. He/ she tips the wheelchair backwards, until the steering wheels reach the curb; he/she pushes the wheelchair straight forward, until the back wheels hit the curb, when he/ she pushes the wheelchair again, until the back wheels go over the curbside.

### Going off the curbside:

The accompanying person positions the wheelchair forwards with their back facing the sidewalk and tips it back until it is balanced. Next, he/ she pushes the wheelchair forwards until the back wheels touch the street. After hurdling the obstacle, he/ she lowers the front steering wheels until they too touch the ground.

### Stairs:

We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that hurdling stairs is more difficult and dangerous.

The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then he/ she pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair.

The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process.

Do not lower the wheelchair until you hurdled the last step of the person in front of the wheelchair.

Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order.

**▲ Do not try to lift the wheelchair at detachable parts (for example: armrests, leg rests and footrests).**

**▲ Avoid using an escalator.**

## LIFTING THE WHEELCHAIR WITH HOISTS

If the wheelchair must be lifted with an articulated crane, to hurdle obstacles, you must not use spots other than the 4 marked with a hook sticker as shown here.

**▲ Do not try to lift the wheelchair at detachable parts (for example: armrests, leg rests and footrests)**

## DRIVING ON AN INCLINE

We suggest avoiding ramps with an incline greater than 9°. There is a chance the wheelchair might tip over in the event of the wheels spinning or riding tilted.

### Ascending a ramp:

Lean your torso forward and use the hand rims with fast strokes, to keep up the proper speed. If you need a brake or relax on your journey, you must make sure to activate the two fixing brakes at the same time.

### Descending a ramp:

Lean your torso backwards and let the hand rims go (flow through your hands), making sure you have the speed and direction under control.

**▲ Avoid sudden directional changes, never try to ascend or descend a ramp diagonally and never try to turn the wheelchair while ascending.**

**▲ The wheelchair is not applicable as a seat for cars.**

## OVERHEATING IN THE SUN OR RADIATORS

The wheelchair must not be permanently exposed to temperatures greater than +40°C. Otherwise the frame as well as seat, backrest and armrests can heat up, which might cause burns.

## Assembly

### Attach wheels and unfold

Attach the wheels by pressing the unlock button in the middle of the hub and insert the wheel into the designated socket on the base. Please make sure that the wheels are properly locked before use, by pulling the wheel towards you away from the wheelchair. (Picture 1+2).

Unfold the wheelchair by pushing the seat until it locks in place.

### Mounting footrests

Mount both footrests as shown in 3 and move them to the inside until you perceive a clear audible snap. Pull the lever upwards and move the footrest to the outside, in order to release/ remove them. (Picture 3+4).

### Adjusting length and inclination of footrests

#### ADJUSTING LENGTH (Abb. 5):

Open and remove screw on the back side of the footrest with the allen key and adjust the length to fit your needs.

Check where the threaded hole is and install the locking screw.

**⚠ Tighten screws!**

#### ADJUSTING INCLINATION (Abb. 6):

Undo the screw on the side of the footrest (plate) with an allen key and pull it out of the gearing.

Then push the plate back into the gearing after adjusting the inclination.

**⚠ Tighten screws!**

### Adjusting seat height

The height of the seat can be adjusted in 3 positions.

Remove both wheels by pushing the release button in order to adjust the height..

Open the nut on the wheel receiver and move it to the desired drilling. Install the nut with the lock washer and tighten it. (Picture 7).

**⚠ Make sure the nut is securely tightened!**

### Adjusting sitting height (front wheels)

The sitting height in the front can be adjusted in 3 positions as well as the desired inclination. (Picture 8).

Remove the screw from the axle in order to adjust.

The plugs in the holes are easily replaced.

Adjust the height and tighten the screw.

**⚠ Make sure the axle is securely tightened!**

### Adjusting forerun of the front wheels

Please loosen the two safety screws in order to adjust the forerun of the front wheels.

The forerun can be adjusted by turning right or left. (Pic. 9).

**Screws after adjustment absolutely firmly tighten and the check the correct holding!**

## Adjusting seat depth

When delivered, the seat depth of all models is 42 cm. The depth can be adjusted in 3 positions up to 47 cm.


Please open the Velcro straps (Picture 10) and fold the front piece of the cushion forward. (Picture 11).

In order to adjust the seat depth, please remove the two screws on the front of the tube on both sides (Picture 12).

You can adjust the seat depth by pulling it forward 3 times to the next drilling, then insert the screw and tighten them. (Picture 13).

**▲ Make sure all screws are installed and tightened!**


Fold the seat cushion back and fasten the Velcro straps.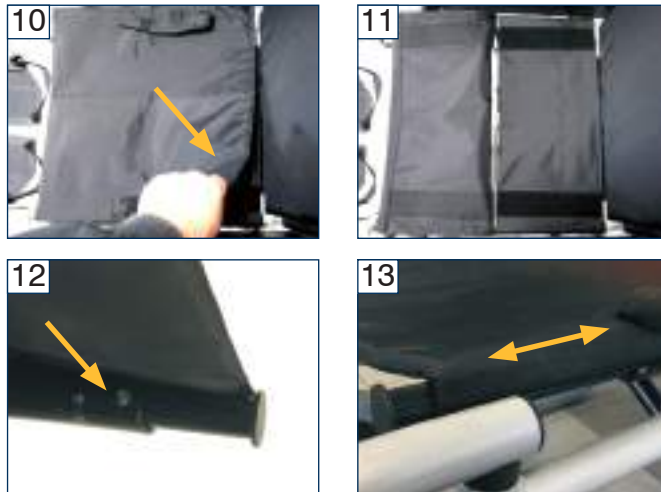

## Adjusting backrest and cushion

The degree of hardness can be individually and variably adjusted by the 5 Velcro fasteners (loosen and tighten) (Picture 14).

In order to mount the back cushion, please lay the cushion on the wheelchair as depicted (Picture 15) and unfold it. Part 1 goes on the inside of the backrest, part 2 on the outside (back), and the 3th part goes under the seat. This will be fixed automatically by the integrated Velcro straps. (Picture 16+17).

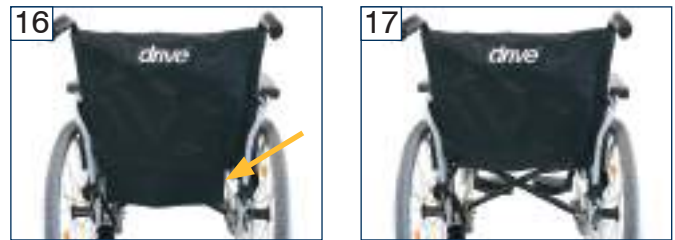## Adjusting height of backrest

The height of the backrest may be adjusted in 3 positions from 42 cm – 46 cm.

**Please note:** When adjusting the height of the backrest, the push handles are adjusted as well.


Loosen the knurled nut on the left and right side (Picture 18), on the bottom of the guide tube attached to the backrest. Remove the screws. Adjust the height by pushing or pulling the push handles into the correct position. Install the screws and tighten the knurled nut.

**▲ Make sure, that both sides are adjusted correctly and on the same height.**

## Adjusting armrest

When delivered, the armrest are already mounted to the wheelchair. The armrests are replaceable.

Depending on your requests, the armrests can be set as short or long armrests.

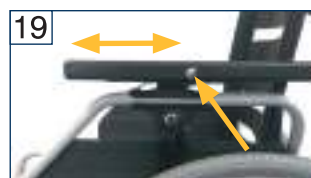To set, press the spring button in the middle on the outside of the armrest (Picture 19) you can now push the armrest into the desired position. (Picture 19).

**Tighten the screws again!**

The armrest can also be adjusted in height by pressing the spring button in the middle under the armrest (Figure 20) you can now pull the armrest up and / or down until you reach the desired position.

**▲ The spring button must lock into position after adjustment. Select the next position if necessary.**

## Swinging away the armrest

To fold back the armrest, you must push the lever on the front of the armrest and pull back the armrest at the same time. (Picture 21).

## Adjusting and using hand brake (optional)

When delivered, the hand brake is adjusted properly for the accompanying person. If the braking effect isn't strong enough the adjusting screw may be loosened or tightened for less braking effect, loosen the lock nut before adjusting the braking effect as explained. Make sure to tighten the lock nut after adjusting, hold the adjusting screw with pliers. (Picture 22).

In order to brake the wheelchair, the accompanying person must pull the braking handles towards the pushing handles.

You can lock the braking handles by pushing the lock flap down with your finger until it locks, in order to lock the brakes for example: for parking, to unlock and release the brakes, simply pull the flap back. (Picture 23+24).

**⚠ If you want to get up from the wheelchair you must activate the fixing brakes additional to the hand brake.**

## Activate parking brakes and use

When delivered, the fixing brakes are correctly adjusted to the configuration condition. If you changed the position of the wheel, the position of the brake might need to be adjusted as well.

To do so, remove the screws on the bottom of the brake (see picture 25) and move the whole unit, so that the space between the pivot bolt and the wheel is as big as 10 mm (see picture 26) Tighten the two screws for an optimal setting.


**⚠ Make sure, that the screws are inserted and tightened properly!**

Push the brake lever forwards in order to apply and activate the brakes until it engages. Pull the lever towards you in order to loosen the brake. (Picture 27+28).

**⚠ Make sure the fixing brakes are locked by moving the wheelchair.**

**⚠ For a comfortable transfer from and to the wheelchair, the parking brakes can be folded. To do so, pull the lever upwards and then fold it. (Picture 29+30).**

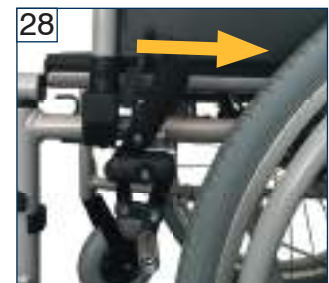## Using the tilting aid

To manoeuvre around kerb stones and over thresholds step heavily (with the left or right foot) on the tilting aid whilst simultaneously pushing down the handles. This ensures that lifting the front of the wheelchair is easier. Ensure that you are standing securely.

## Using the anti-tippers

By loosening the lock, the anti-tippers can be adjusted in height.

## Folding

You can fold this wheelchair with only one pull, for easy transportation and storage in a car.

To do so, please remove the optional seat cushion and footrests first, then pull up the straps attached to the seat – done!

## Cleaning / maintenance / reuse

- Clean regularly with moist cloth.
- Don't use any aggressive cleaning agents.
- You may use a sprayable disinfectant, for example: Descosept AF or Bacillol AF
- Check screws regularly and retighten if necessary
- Check condition of wheels regularly, these are durable and unbreakable -> air pressure must not be checked and refilled. Please DO check if the tire is scratched, torn or damaged.
- If broken, please contact your DRIVE MEDICAL specialized dealer.
- This product can be reused. When preparing, you must regard the manufacturer's standards.

## Warranty

We issue a warranty of 24 months from date of purchase.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, handgrips, and brakes which are subject to normal wear and need periodic replacement.

Subject to technical alterations. Errors and omissions excepted.

## Service of Life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

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